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NL

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**The advantages at a glance:**

**GB**

- Allows practical and effective care of superficial scalds and minor burns of 1<sup>st</sup> and 2<sup>nd</sup> degree
- Pleasantly cooling and pain relieving
- Promotes wound healing
- Reduces the risk of blister formation
- Almost invisible
- Waterproof
- Provides an optimal moisture balance in the wound and thus enables a quick healing process
- The outer film is breathable and skin-friendly
- Allows visual inspection of the wound healing process due to its transparency
- Ensures an optimal and comfortable hold on the skin due to the extra thin material
- Does not stick to the wound
- Provides good mechanical protection due to the cushioning effect of the gel
- Prevents the penetration of dirt and bacteria
- Can remain on the wound for up to 7 days

**The principle of moist wound healing:**

The adhesive burn plasters consist of a moist gel. They are able to release moisture and take up exudate at the same time. The innovative principle of moist wound healing actively supports the body's natural healing mechanisms and enables a quick and uncomplicated healing process (Figure 1).

Scientific studies have shown that skin cells in a

moist wound environment reproduce more effectively and fuse together more easily, thereby promoting wound healing.

**Please note:**

- The burn plasters must not be used on infected wounds.
- Please consult a physician if:
  - the wound shows signs of an infection (severe reddening, overheating, swelling or pain)
  - the wound does not begin to heal within 4 days
  - for the treatment of large surface burns

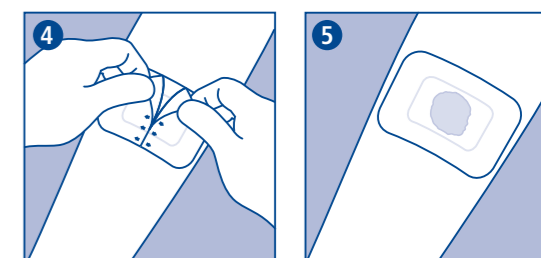
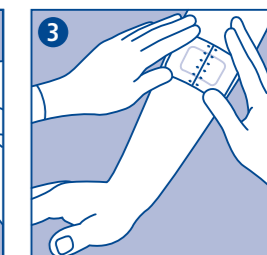
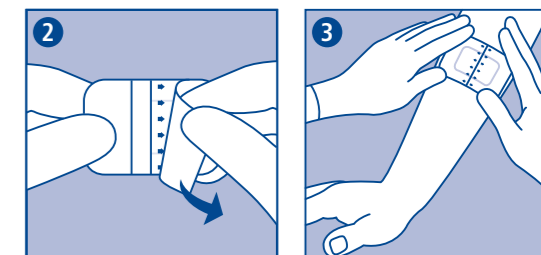
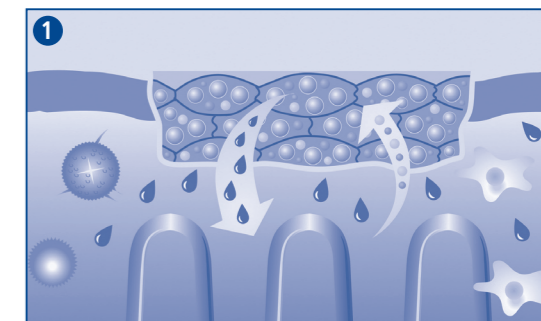
The burn plaster may occasionally cause skin reactions in susceptible subjects.

**Instructions for use:**

- Wash hands before and after wound treatment.
- Where possible cool the affected area for approx. 5 minutes under cold running water. Then dry the wound surroundings carefully with a clean, lint-free and, where possible, sterile material.
- Next remove a strip of the white protective backing from the underside of the plaster (Figure 2).
- Lay the burn plaster with the exposed side on the wound and then remove the remaining protective backing.
- Press the plaster down evenly and particularly firmly around the edges (Figure 3).
- Carefully remove the thin, upper protective backing (Figure 4).

- The wound healing process can be monitored easily without removing the dressing. (Figure 5).
- The burn plasters should be used until the wound has completely healed.
- To remove the plaster, lift it at one edge and remove the whole plaster carefully.
- Store plasters in a cool, dry place. Keep out of the reach of children.
- Contents sterile unless peel packs are damaged.

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**PAUL HARTMANN AG**
- BE – N.V. PAUL HARTMANN S.A.
- CH – IVF HARTMANN AG
- GB – PAUL HARTMANN Ltd.
- IT – PAUL HARTMANN S.p.A.
- NL – PAUL HARTMANN B.V.