Re: Údržba AED HeartSine

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05 December 2017

Dear Valued Customer,

All HeartSine AEDs perform a comprehensive weekly self-test that will alert the device owner if the device requires service. The LED status indicator on the front of the AED flashes green when the device has passed its weekly self-test. If the device requires service, the indicator light will flash red in tandem with an audible "beep" to draw attention to the device status.

The User Manual for our AEDs recommends that regular visual inspection of the status indicator is performed on a weekly basis. It should be noted whilst this is a recommendation, it is not a mandate on the part of the device owner. We would suggest at the very least, that a monthly visual inspection of the device is a viable option to ensure device readiness.

There is no annual inspection of AEDs required.

Many thanks for your consideration of HeartSine products.

Regards,

A handwritten signature in black ink, appearing to read "Peter O'Hare".

Dr Peter O'Hare

**Global Product Manager
HeartSine AEDs
Physio-Control, now part of Stryker**