AEROresc® - EASY Collar Adult

AEROresc® - EASY Collar Mini

de Deutsch

en English

fr Français

es Español

it Italiano

pt Português

nl Nederlands

sv Svenska

1. Die Modelle

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1. The models

REF: HMT11-EC-K
Article description: AEROresc® - EASY Collar Mini, adjustable cervical collar for infants, children and small adults, color blue
REF: HMT11-EC-E | HMT11-EC-T (Tactical)
Article description: AEROresc® - EASY Collar Adult, adjustable cervical collar for adults, color yellow (HMT11-EC-E), color green tactical (HMT11-EC-T)

2. Intended purpose and description

This extraction collar is a one-piece rigid cervical spine immobilization device designed to assist the rescuer with the maintenance of neutral alignment, prevention of lateral (side-to-side) sway and anterior-posterior (forward and backward) flexion and extension of the cervical spine during transport and routine patient care or movement. The AEROresc® - Easy Collar Adult can be adjusted to 16 sizes, the AEROresc® - Easy Collar Mini can be adjusted to 12 sizes. Both models are radio-transparent by CT and MRI.

3. Principles of immobilization

Prevention and/or further extension of damage to the cervical spine and spinal cord is accomplished by maintaining the head and neck in a position of "neutral alignment". The head and neck should not be flexed forward, extended backward, or cocked to one side or the other. The neutral alignment position prevents damaged vertebrae or foreign bodies from rubbing and/or cutting the spinal cord. Additionally, the neutral alignment position prevents the spinal cord from being twisted or compressed causing physical and/or ischemic injury to the spinal cord.

Warnung

This collar is intended for application by licensed health care practitioners as directed by a physician or other medical professional. Permanent injury may result from use other than as directed by a physician or other medical authority.
An extraction collar is only part of total patient immobilization. It is important that practitioners are trained and practiced in the use and application of the device and that all instructions are read and understood prior to application of the collar. All directions are to be considered as guidelines and not an attempt to define medical practice.

4. Directions for use

1. Have one rescuer manually stabilize the head and neck, gently holding the head and neck in the neutral alignment position (see Figure 1).
NOTE: If resistance is felt or an increase in patient discomfort is noted, follow your institution's protocols for immobilization of the cervical spine in the presence of known or suspected cervical dislocation.

2. Once the head and neck are manually stabilized, a second rescuer should apply an extraction collar to the patient. Size the collar as determined below. Apply the collar must not hyper-extend patient's neck.

3. Measure the distance between an imaginary plane drawn horizontally and immediately below the patient's chin and second horizontal plane drawn immediately on top of the patient's shoulder (see Figure 2).

4. Compare this distance with the distance from the collar sizing line to the other end of the sizing label and read the right size number (see Figure 3).

5. Sizing: The AEROresc® - Easy Collar Adult is preset to Neckless Size 1. The AEROresc® - Easy Collar Mini provides 12 ratchet settings between the traditional sizes Infant and Tall. Simply pull the collar apart until the size number on the windows equals the user's finger measurement (Figure 5).

6. Engage the safety locks by pushing down on the locking buttons (Figure 6).

5.1 Application of the cervical collar
Proper application of any extraction collar requires two individuals. The first rescuer should maintain stabilization and neutral alignment of the head and neck by grasping the head with a spread finger technique (see Figure 1).

5.2 The second rescuer should stabilize the front of the collar along the patient's chest and position the chin piece. The collar body should rest on the top of the patient's shoulder and against the sternum without gaps (see Figure 7). The patient's chin must rest securely in the chin piece in neutral alignment position (see Figure 8).

5.3 While holding the front of the collar in place, wrap the back of the collar around the back of the patient's head and neck and secure to the front of the collar with the velcro tab (see Figure 8). Ensure that the velcro tab is firmly attached and parallel to the fixed hook velcro tab on the front of the collar.

5.4 Adjust the velcro tab as necessary by holding the front cut out of the collar and tightening the velcro tab (see Figure 9).

Warnung

Maintain neutral head and neck alignment during the entire procedure. Once the collar is properly applied and the patient is immobilized, the head should not be released (see Figure 10).
Further immobilize the head and neck with head blocks or a head immobilizer and secure the patient to the backboard with appropriate retention straps.

Supine application

6.1 If the patient is supine, slide the back portion of the collar directly behind the patient's neck. Once the back portion of the collar is evenly positioned behind the head, the front portion of the collar can be positioned under the chin (see Figure 11).
6.2 While maintaining the proper positioning of the collar with one hand, attach the velcro with the other hand (see Figure 12). Ensure that the velcro tab is firmly attached and parallel to the fixed hook velcro tab on the front of the collar.

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