### Instructions For Use

The Piranha Trauma Shears are intended to assist with the removal of clothing to allow access to wound areas and to cut gauze as required for treatment. To use product, insert thumb into the small hole of the shears and middle finger and ring finger into the larger hole of the shears. When cutting clothing, place the shears with the blunt folded tip toward the patient. Use the free hand to grasp the clothing and assist in the cutting process. To cut with the shears, use the fingers to open and close the shears as the cutting hand applies forward pressure to the shears. Only reuse shears after they have been properly cleaned (see cleaning and storage section). Replace shears when product becomes dull or no longer functions as intended. Shears are NOT STERILE and are not intended for use in cutting live tissue.

Note: If you need additional guidance to use this product, please contact a healthcare professional. Any serious incident that occurs in relation to the device shall be reported to the manufacturer and to the competent authority in which the user/patient is established.

### Cleaning and Storage

Per manufacturer's recommendation, after removing visible contaminants from the device, use an alcohol, chlorine, or hydrogen peroxide based cleanser, ensuring wet contact times are appropriate for the contaminant. Dry shears when clean.

Store Piranha Shears in a clean and dry condition.

### Limited Warranty

The products and components contained in this package are medical devices intended solely for use as instructed. Tytek Medical Inc. warrants that the products and components are free from defects in material and workmanship and fit solely for use as medical devices for the specific purpose described in the manual for a period of 12 months from the date of shipment by TyTek Medical Inc. Any use of the products or components beyond the specific use described in this manual, voids this limited warranty. EXCEPT AS SET FORTH HEREIN, TYTEK MEDICAL INC. MAKES NO OTHER WARRANTIES, EXPRESS OF IMPLIED, REGARDING THE PRODUCTS OR COMPONENTS AND SPECIFICALLY DISCLAIMS THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

This item sold and to be used strictly per terms and conditions at www.tytekmedical.com

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